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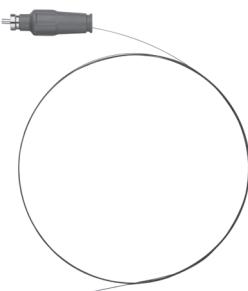
Does not apply to USA and Canada!
US and Canadian customers, please use the separate US instructions!

1 Article numbers

UL004, UL005, UL006

2 Intended use

The optical fibers (laser fibers) are designed for use in connection with the Calculase, Calculase II and Calculase III laser units from KARL STORZ in the wavelength spectrum of 2100 nm.
They are intended for use in operating rooms and doctors' offices for human lithotripsy applications and for the cutting, ablation and vaporization of tissue.

2.1 Indications

The use of single-use laser fibers is indicated if the following is indicated in the opinion of the attending physician.

Lithotripsy:

Stones of all types in the ureteral system (urethra, ureter, bladder and kidneys), bile duct, gall bladder, salivary duct and salivary glands

Urology:

- Holmium laser ablation – HoLAP
- Holmium laser resection of the prostate – HoLRP
- Holmium laser enucleation of the prostate – HoLEP
- Obstruction of the bladder neck
- Prostate and bladder incision
- Hemostasis, vaporization and excision for treatment of benign prostatic hyperplasia
- Superficial and invasive bladder, urethral and ureter tumors
- Condylomata
- Strictures and stenoses in the urinary tracts
- Ureteroscopic laser papillotomy

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**Calculase III Laser fiber
(single-use laser fibers)**

- Laser endoureterotomy
- Laser endopyelotomy
- Laser urethrotomy
- Urothelial tumors of the upper urinary tracts (UUT-UT)
- Treatment of ureteropelvic junction obstruction
- Multitrac MPCNL
- Primary or refractory urethral stricture with holmium laser endourethrotomy (HLU)

2.2 Contraindications

No contraindications relating directly to the product are currently known.
The laser fiber must not be used if, in the opinion of the responsible physician, the health of the patient is endangered through such use, e.g., due to the general condition of the patient.

Improper use, reprocessing and multiple use of the laser fiber transmission (other than the intended use) is prohibited for safety reasons.

Unauthorized conversions or modifications to the laser fiber are not permitted for safety reasons.

3 Safety instructions

WARNING: Risk of infection and injury: Only use the laser fiber if the packaging is intact and the expiry date has not been exceeded. Observe the recommended storage time for medical devices.

CAUTION: Risk of material damage when inserting the laser fiber into endoscopes. Only insert the laser fiber when the working channels are straight. No laser light emission inside the endoscope.

WARNING: Risk of injury: Be sure to wear protective glasses tailored to the laser used. To this end, compare the details on the protective glasses with the details in the manual for the relevant laser unit.

WARNING: Risk of injury: Damaged laser fibers can lead to injuries to the patient. Laser fibers and accessories used in combination must be checked immediately before each application to make sure they are complete, free from damage and in full working order. Broken laser fiber parts must not be left inside the patient.

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WARNING: Risk of fire: Laser light from Class 4 lasers represents a source of ignition. When using the laser fiber, take suitable measures to prevent laser-induced fires and explosions.

WARNING: Risk of infection and injury: This product is sterile on delivery and is only intended for single use. Laser fibers are disposable products, the special design and material properties of which mean that they may only be reprocessed and used once.

WARNING: Avoid extreme bending or tight winding of the fiber. This could cause the fiber to break. The minimum bending radius is between 1 and 3.5 cm, depending on the fiber (230 µm fiber: 1.5 cm, 365 µm fiber: 2.0 cm, 600 µm fiber: 3.5 cm). Ensure that this is also taken into consideration for storage.

WARNING: Risk of infection and injury: After opening the packaging, a laser fiber which is not used must be disposed of and a new fiber used.

WARNING: Risk of injury if the maximum output and energy of the laser fiber is exceeded. Only use laser fibers with the specified laser units.

Art. no.	E _{max} [J]	P _{max} [W]
UL004	2	20
UL005	4	35
UL006	4	35

4 Handling

It is recommended that the suitability of the devices for the intended procedure be checked prior to use.

- Note the expiry date.
- The original packaging must not be opened until immediately before the product is used.
- Check the packaging for damage: Damaged packaging may compromise the sterility of the product.
- When removing the product from the primary packaging, the rules of aseptic handling must be followed.
- The product must be examined for damage such as cracks, deformations, etc. Defective products must not be used under any circumstances.

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Calculase III Laser fiber (single-use laser fibers)

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1. In this case, check the proximal laser fiber surface and trim the laser fiber.

If you are still unable to achieve a round aiming beam, the laser fiber must be replaced if treatment results prove inadequate.

5 Description of the fiber

Depending on the fiber type, various pulse energies and medium laser outputs can be applied (see instruction manual of the Calculase, Calculase II or Calculase III laser devices).

5.1 Trimming the fiber

The fibers are quartz fibers with a protective plastic exterior. They can be trimmed intraoperatively.

1. Remove the protective jacket using the respective fiber stripper from the fiber trimming set.
2. Trim the tip of the fiber using the ceramic knife (see Calculase I – III instruction manuals).

CAUTION: Risk of material damage: Use the sterilizable KARL STORZ fiber trimming set when handling laser fibers for intraoperative reprocessing of the distal end of the laser fiber.

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6 Application/storage conditions

The product must be stored in a clean and dry place away from direct sunlight and must not be used after the expiry date.

	Temperature	Relative humidity
Storage	-20 °C...+60 °C	10 %...90 %
Application	+10 °C...+40 °C	30 %...70 %

7 Disposal

For disposal, no special measures are necessary. Disposal must be carried out in accordance with the respective applicable local and national laws and regulations.

8 Directive compliance

This medical device bears a CE mark in accordance with the Medical Devices Directive (MDD) 93/42/EEC. A code number after the CE mark indicates the responsible notified body.

9 Repairs and modifications

The performance of any repairs or modifications to the instrument by unauthorized persons is not permitted and shall release the manufacturer from any liability.

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10 Symbol description

Symbol	Meaning
	Manufacturer
REF 26008 KA	Special symbols – passivation. Passivated goods are identified through the underlining of the article number.
LOT	Batch code
QTY	Number of products in the product packaging
	Keep dry
RxOnly	In accordance with US federal law (21 CFR 801.109), this product may only be sold on prescription from a licensed physician.

11 Packaging symbols

For the meanings of the symbols printed on the packaging, please refer to the 'Packaging symbols' accompanying instruction leaflet, mat. no. 96216316DF. It can be downloaded from www.karlstorz.com.

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REF UL004, UL005, UL006

2 Intended use

The optical fibers (laser fibers) are designed for use in connection with the Calculase, Calculase II and Calculase III laser units from KARL STORZ in the wavelength spectrum of 2100 nm. They are intended for use in operating rooms and doctors' offices for human lithotripsy applications and for the cutting, ablation and vaporization of tissue.

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Urology:

- Holmium laser ablation – HoLAP
- Holmium laser resection of the prostate – HoLRP
- Holmium laser enucleation of the prostate – HoLEP
- Obstruction of the bladder neck
- Prostate and bladder incision
- Hemostasis, vaporization and excision for treatment of benign prostatic hyperplasia
- Superficial and invasive bladder, urethral and ureter tumors
- Condylomata

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- Strictures and stenoses in the urinary tracts
- Ureteroscopic laser papillotomy
- Laser endoureterotomy
- Laser endopyelotomy
- Laser urethrotomy
- Urothelial tumors of the upper urinary tracts (UUT-UT)
- Treatment of ureteropelvic junction obstruction
- Multitrac MPCNL
- Primary or refractory urethral stricture with holmium laser endourethrotomy (HLU)

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2.2 Contraindications

No contraindications relating directly to the product are currently known.

The laser fiber must not be used if, in the opinion of the responsible physician, the health of the patient is endangered through such use, e.g., due to the general condition of the patient.

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4.1 Removing the laser fiber from the packaging

1. Open the sterile packaging.
2. Under aseptic conditions, carefully remove the laser fiber together with the laser fiber carrier from the sterile packaging.
3. Remove the distal laser end from the carrier and carefully unwind the laser fiber from the carrier.
4. Remove the connector from the holder on the carrier.

CAUTION: Risk of material damage: Do not touch the light entrance surface.

CAUTION: Risk of material damage: Do not damage the proximal light fiber end.

NOTE: Avoid contact with the fiber end and do not soil the fiber end.

4.2 Plugging the connector into the laser device

1. To do so, plug the connector axially into the fiber socket on the laser device.
2. Fix the connector with the nut and tighten by hand.

CAUTION: Risk of material damage: Do not tilt the laser fiber connector. Do not use tools.

4.3 Functional test

Check optical laser fiber transmission.

1. Switch the laser device on.
2. Connect the laser fiber.

The aiming beam is now available.

Calculase = red,

Calculase II and Calculase III = green.

3. Check the distal reproduction of the aiming beam light.

Optical laser fiber transmission is OK

Optical laser fiber transmission is **OK** if the aiming beam produces a round, evenly bright mark at a distance of 5 to 10 cm from the distal fiber end.

Optical laser fiber transmission is not OK

Optical laser fiber transmission is **not OK** if the mark produced is not round, is unevenly bright or is extremely irregular (comet tail).

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- In this case, check the proximal laser fiber surface and trim the laser fiber.
- If you are still unable to achieve a round aiming beam, the laser fiber must be replaced if treatment results prove inadequate.

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- Remove the protective jacket using the respective fiber stripper from the fiber trimming set.
- Trim the tip of the fiber using the ceramic knife (see Calculase I – III instruction manuals).

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NOTE: Stripping length 10 – 15 mm.

5.2 Removing the laser fiber

- Switch the device to standby mode.
- Remove the laser fiber.
- Dispose of the fiber for single use.

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