

SciCan Ltd.
1440 Don Mills Road
Toronto, Ontario
M3B 3P9
Canada

T +1 416-445-1600
F +1 416-445-2727
www.scican.com

ADDENDUM FOR OPHTHALMOLOGY INSTRUMENTS 95-114834 STAT/IM 2000/5000 G4 Operator's Manual

The purpose of this addendum is to include instruments commonly used for Ophthalmology that will be sterilized in the STAT/IM 2000/5000 G4.

5.2.1. Unwrapped Cycle (STATIM 2000 G4)

The Unwrapped Cycle is a general purpose sterilization cycle used to sterilize up to 1.0 kg (2.2 lbs.) of solid metal instruments such as pliers, burrs, scalers and forceps. Dental handpieces and Phacoemulsification (hollow/lumen) instruments may be sterilized in this cycle.

To select the Unwrapped Cycle, press the Unwrapped Cycle button, then press the START button.

5.3.1. Unwrapped Cycle (STATIM 5000 G4)

The Unwrapped Cycle is used to sterilize light loads (less than 0.5 kg or 1.1 lbs.) of solid metal instruments such as pliers, burrs, scalers, and forceps. Phacoemulsification (hollow/lumen) instruments may be sterilized in this cycle.

To select the Unwrapped Cycle, press the Unwrapped Cycle button, then press the START button.

5.3.4. Heavy Duty Unwrapped Cycle (STATIM 5000 G4)

The Heavy Duty Cycle is used to sterilize larger loads of unwrapped metal instruments weighing up to 1.5 kg (3.3 lbs.). Dental hand pieces and Phacoemulsification (hollow/lumen) instruments can be sterilized in this cycle. Using the STAT/IM 5000 Extended Cassette (order no. 01-112509S) will allow for the sterilization of rigid endoscopes.

To select the Heavy Duty Unwrapped Cycle, press the Heavy Duty Unwrapped Cycle button, then press the START button.